

PRECAUTIONS AND LIMITATIONS

- Do not use this test past the expiration date.
- Make sure the package is sealed and do not open it until you are ready to use the test.
- Certain health conditions, such as an ovarian cyst or ectopic pregnancy (pregnancy outside the uterus), can cause a false or irregular result.
- Procedures should be followed precisely for accurate results.
- All results should be confirmed by your healthcare provider, especially when making decisions about future medical care.

To increase your chances for a healthy pregnancy for you and your baby, you should consider the following:

- Maintain a well-balanced diet.
- Do not smoke

The period from conception through the third month of pregnancy is a very important time. You should see your doctor regularly throughout your pregnancy for proper care and nutritional counseling.

WARNING:

According to the Surgeon General, women should not drink alcoholic beverages during pregnancy. Drinking alcohol creates the risk of birth defects.

If you have any questions about the instructions or your results, call our Help Line Monday – Friday between 7:00 a.m. – 5:00 p.m. Eastern Time. Toll-Free 1-800-367-6022, or visit us at [www.FirstResponse.com](#)

1FIRSTRESPONSE™PREGNANCY

Manufactured for Church & Dwight Co., Inc. Ewing, NJ 08628

FIRST RESPONSE is a trademark of Church & Dwight Co., Inc.

[www.FirstResponse.com](#)

CAUTION:

THIS PRODUCT CONTAINS A LITHIUM COIN BATTERY.

- Do not disassemble device and dispose according to local regulations.
- This battery is non-replaceable, hazardous, and must be kept away from children.
- This battery can cause severe or fatal injuries in 2 hours or less if it is swallowed or placed inside any part of the body.
- Seek immediate medical attention if it is suspected the battery has been swallowed or placed inside any part of the body. Contact the 24-hour National Battery Ingestion Hotline (1-800-498-6666), or a doctor at once.

Need Spanish? Scan here

72031512

FRIN-90140-12

1FIRSTRESPONSE™PREGNANCY

Watch How To Take the Test Here

Digital Pregnancy Test

When to use the FIRST RESPONSE™ Digital Pregnancy Test

FIRST RESPONSE™ Digital Pregnancy Test is designed to detect hCG (human chorionic gonadotropin) as early as 6 days before your missed period (5 days before the day of your expected period).

FIRST RESPONSE™ Digital Pregnancy Test is more than 99% accurate when used from the day of your expected period. You can use the FIRST RESPONSE™ Digital Pregnancy test at any time of day. You do not have to use first morning urine. However, your first morning urine generally contains the highest level of hCG, the pregnancy hormone, and is recommended if you are testing early.

PLEASE READ ALL THE INFORMATION IN THIS INSERT BEFORE PERFORMING THE TEST.

WARNINGS AND PRECAUTIONS

For in-vitro diagnostic use. Not to be taken internally. Do not use after the expiration date stamped on the bottom of the carton. Single-use device. Do not reuse. Keep out of the reach of children. Do not disassemble the device. Do not eat any component of this device. If accidentally swallowed, seek medical attention.

STORAGE

Store the FIRST RESPONSE™ Digital Pregnancy Test in a dry place below 86°F (30°C). Do not freeze.

BEFORE YOU BEGIN:

THIS DEVICE SELF-ACTIVATES UPON REMOVAL FROM THE WRAPPER. DO NOT OPEN WRAPPER UNTIL YOU ARE READY TO USE. After reading the insert when you are ready to perform the test, remove the test stick from the foil wrapper and remove the Overcap. (Upon removal from the wrapper, check the Display Screen for the presence of a steady clock symbol ⌚. The device is ready for use. Do not use the test unless the steady clock symbol is on the Display Screen.)

Overcap

Absorbent Tip

Display Screen

Thumb Grip

INSTRUCTIONS (How to use):

STEP 1

Test In Your Urine Stream

- Hold Thumb Grip so that the Absorbent Tip is pointing downward and the Display Screen is facing away from your body.
- Hold the Absorbent Tip in your urine stream for 5 seconds. Be careful not to get the urine on the plastic holder. Throughout testing, never hold the device with the Absorbent Tip pointing upward.

OR

Test In Container

- To use the dip method, collect your urine in a clean, dry cup.
- Dip the entire Absorbent Tip in the urine for 5 seconds.

5 SECONDS

STEP 2

INSTRUCTIONS (How to use):

STEP 2

- While still holding the stick with the Absorbent Tip facing downward, replace the Overcap and lay the stick on a flat non-absorbent surface with the Display Screen facing up.
- The clock symbol ⌚ will begin to blink in about 30 seconds after urine is applied to indicate the test is working.

Test is working

WAIT

- Wait 3 FULL MINUTES for the test results to develop and a result to be displayed.

READING THE RESULTS

- In approximately 3 minutes, an unmistakable “YES+” or “NO-” will appear on the Display Screen.

PREGNANT

If you see “YES+” the test indicates pregnancy.

NOT PREGNANT

If you see “NO-”, you are likely not pregnant.

72031512

FRIN-90140-12

QUESTIONS AND ANSWERS

Q: How does the test work?

A: The FIRST RESPONSE™ Digital Pregnancy Test detects hCG (human chorionic gonadotropin) your body makes during pregnancy. The hormone is found in urine when pregnant.

Q: When can I test with FIRST RESPONSE™ Digital Pregnancy Test?

A: The FIRST RESPONSE™ Digital Pregnancy Test is designed to detect hCG as early as 6 days before your missed period (5 Days before the day of the expected period). The amount of pregnancy hormone increases as pregnancy progresses over approximately the first three months. In laboratory testing with early pregnancy urine samples, FIRST RESPONSE™ Digital Pregnancy Test gave the following results.

NUMBER OF DAYS BEFORE THE EXPECTED PERIOD	% OF SAMPLES FROM PREGNANT WOMEN GIVING A POSITIVE RESULT
1	>99%
2	>99%
3	96%
4	86%
5	60%

Q: How accurate is the test?

A: FIRST RESPONSE™ Digital Pregnancy Test is >99% accurate from the day of the expected period in laboratory studies. However, some pregnant women may not have detectable amounts of pregnancy hormone in their urine on the day they test or the first day of their period may be later than anticipated.

Q: What do I do if the result on the Display Screen is “YES+” (pregnant)?

A: Remember, this test is not intended to replace your healthcare professional’s diagnosis. A “YES+” test result shows that the pregnancy hormone (hCG) was detected, which normally indicates pregnancy. You must see your doctor to confirm that you are pregnant. If you test positive, but think you may not be pregnant, you should check with your doctor.

Q: What do I do if the result on the Display Screen is “NO-” (not pregnant)?

A: You may not be pregnant, or it may be too early to tell. If you do not get your period within a few days, you should retest with another FIRST RESPONSE™ Digital Pregnancy Test. It is possible that your expected period may be later than anticipated or your urine may not have had enough pregnancy hormone for the test to register a “YES+” result. If upon retesting you get another “NO-” result and your period has still not started, you should call your doctor.

Q: The blinking clock ⌚ has disappeared followed by a “?” on the Display Screen. What does this mean?

A: A “?” symbol indicates an error has occurred during testing. You should retest with another FIRST RESPONSE™ Digital Pregnancy Test, carefully following all directions.

Q: The Display Screen is blank. What is wrong?

A: If the display screen goes blank at any time before yielding a result, an error has occurred. You should retest with another FIRST RESPONSE™ Digital Pregnancy Test, carefully following all directions.

Q: How long will my result show on the Display Screen?

A: Your result will be displayed for at least 30 minutes. Read your results within this time frame. Once you have read your result, discard the stick.

Note: FIRST RESPONSE™ Digital Pregnancy Test Sticks contain a non-removable lithium-coin battery. Be sure to dispose of the digital test stick according to local regulations regarding the disposal of lithium batteries.

Q: What will happen if the plastic holder gets wet?

A: Some splashing while using the test will not damage the test, but if it becomes very wet it can become damaged. If the test becomes damaged, the Display Screen will either go blank or an error “?” message will appear.

Q: Do any drugs or medications affect the test?

A: Only medications containing the pregnancy hormone, human chorionic gonadotropin (hCG), can affect the result. The test should not be affected by hormone therapies containing Clomiphene citrate, alcohol, pain killers, antibiotics, or the contraceptive pill.

Q: I removed the device from the wrapper and the clock symbol appeared on the Display Screen; however, I’m not ready to use the test. What should I do?

A: Once removed from the wrapper, the device must be used within 12 hours. Keep the device dry before use.

Q: Will the amount of liquid I drink affect the result?

A: Excessive fluid intake including water could dilute your urine, causing the amount of hCG present not to be detected properly. You should not drink too much liquid before testing.