

PRECAUTIONS AND LIMITATIONS

- Do not use this test past the expiration date.
- Make sure the package is sealed and do not open it until you are ready to use the test.
- Certain health conditions, such as an ovarian cyst or ectopic pregnancy (pregnancy outside the uterus), can cause a false or irregular result.
- Procedures should be followed precisely for accurate results.
- All results should be confirmed by your healthcare provider, especially when making decisions about future medical care.

To increase your chances for a healthy pregnancy for you and your baby, you should consider the following:

- Maintain a well-balanced diet.
- Do not smoke.

The period from conception through the third month of pregnancy is a very important time. You should see your doctor regularly throughout your pregnancy for proper care and nutritional counseling.

WARNING:

According to the Surgeon General, women should not drink alcoholic beverages during pregnancy due to the risk of birth defects.

If you have any questions about the instructions or your results, call our Help Line Monday – Friday between 7:00 a.m. – 5:00 p.m. Eastern Time. Toll-Free 1-800-367-6022, or visit us at www.FirstResponse.com

1 FIRST RESPONSE™ PREGNANCY

Manufactured for Church & Dwight Co., Inc. Ewing, NJ 08628
FIRST RESPONSE is a trademark of Church & Dwight Co., Inc.
www.FirstResponse.com

Need Spanish?
Scan below



72031514
FRIN-80005-02

1 FIRST RESPONSE™ PREGNANCY



Watch How To Take The Test Here

EasyCup Pregnancy Test

When to use the FIRST RESPONSE™ EasyCup Pregnancy Test
The FIRST RESPONSE™ EasyCup Pregnancy Test is a rapid chromatographic immunoassay for qualitative detection of human chorionic gonadotropin (hCG) **on the day of your missed period or any day thereafter.**

You can use the FIRST RESPONSE™ EasyCup Pregnancy test at any time of day. You do not have to use first morning urine. However, your first morning urine generally contains the highest level of hCG, the pregnancy hormone.

PLEASE READ ALL THE INFORMATION IN THIS INSERT BEFORE PERFORMING THE TEST.

WARNINGS AND PRECAUTIONS
For in-vitro diagnostic use. Do not use test components beyond the expiration date on bottom of the carton. Make sure the protective pouch is sealed. Each protective pouch contains one (1) cup in a sealed protective pouch with an integrated test strip, and a moisture absorbent pack. The absorbent pack helps to preserve the test strip while enclosed inside the pouch. This pack is not to be used in connection with the test and should be discarded once the pouch is opened. Do not use if protective pouch seal is open, torn or damaged. Keep out of reach of children. For external use only. **Single-use device.** **Do not reuse.** Keep out of sunlight. Do not disassemble the cup and the test strip. Do not eat any component of this device. If accidentally swallowed, seek medical attention.

STORAGE
Store the FIRST RESPONSE™ EasyCup Pregnancy Test at room temperature 38°F - 86°F (4°C - 30°C).

BEFORE YOU BEGIN:
CAREFULLY READ THE INSTRUCTIONS AND DO NOT OPEN THE FOIL POUCH UNTIL YOU ARE READY TO TEST. Failure to do so may result in invalid test results.

- Have a clock or timer.



1 FIRST RESPONSE™ PREGNANCY

Manufactured for Church & Dwight Co., Inc. Ewing, NJ 08628
FIRST RESPONSE is a trademark of Church & Dwight Co., Inc.
www.FirstResponse.com

Need Spanish?
Scan below



72031514
FRIN-80005-02

INSTRUCTIONS (How to use):

STEP 1

- Remove the test cup from its protective pouch by tearing at the “notch”. Use immediately.
- Open the test cup lid. The test strip is already inside of the cup. Do not remove the test strip.

STEP 2

- Urinate directly into the test cup. Fill at least to the minimum mark on the label. Do not overfill the cup with urine.
- Set the test cup on a level surface. Do not shake the cup.

CONTINUED →

INSTRUCTIONS (How to use):

WAIT

- **Wait 5 FULL MINUTES** for the test result to develop and a result to be displayed.
- Pink lines will start to develop. If not, see “Invalid Result” below. Test results should not be read after 15 minutes. The test results are invalid after 15 minutes.

READING THE RESULTS

Test results can be viewed through the cup. The test strip does not need to be removed from the cup to read the test result.

PREGNANT
If you see two pink lines, the test indicates pregnancy. *It does not matter how faint or dark (or how wide) the lines are. One line may be lighter than the other. Appearance of the results may vary.*



NOT PREGNANT
If you see only one pink line, you are likely not pregnant.



INVALID RESULT
After performing the test (5 minutes) and no pink color line is visible within the control window, this result is considered invalid. The test has not worked, possibly because too much or too little urine was used. If no pink color line appears, the directions may not have been followed correctly. You will need to repeat the test by using a new FIRST RESPONSE™ EasyCup Pregnancy Test.

AFTER TESTING

Flush the remaining urine sample in the cup down the toilet. Discard all components in a waste basket. Do not reuse the cup or test.



72031514
FRIN-80005-02

QUESTIONS AND ANSWERS

Q: How does the test work?
A: The FIRST RESPONSE™ EasyCup Pregnancy Test detects a very small amount of hCG (human chorionic gonadotropin) your body makes during pregnancy. The hormone is found in urine when pregnant.

Q: When can I test with FIRST RESPONSE™ EasyCup Pregnancy Test?
A: The FIRST RESPONSE™ EasyCup Pregnancy Test is designed to detect hCG on the first day of your missed period and then any day after. You do not have to use first morning urine although it is highly recommended because if you are pregnant, the first morning urine generally contains the highest concentration of hCG.

Q: When should I read my results?
A: The FIRST RESPONSE™ EasyCup Pregnancy Test results should be read as soon as possible, after waiting 5 minutes. Test results should not be read after 15 minutes. The test results are invalid after 15 minutes.

Q: How accurate is the test?
A: Laboratory tests and consumer studies show the FIRST RESPONSE™ EasyCup Test to be more than 99% accurate at detecting typical pregnancy hormone levels from the day of your missed period. Note hormone levels vary. However, some pregnant women may not have detectable amounts of the pregnancy hormone in their urine on the first day of the missed period or their period may be later than anticipated.

Q: What do I do if the result shows two pink lines (Pregnant)?
A: Remember, this test is not intended to replace your healthcare professional’s diagnosis. A positive (pregnant) test result shows that the pregnancy hormone (hCG) was detected, which normally indicates pregnancy. You must see your doctor to confirm that you are pregnant. If you test positive, but think you may not be pregnant, you should check with your doctor.

Q: What do I do if the result shows one line (Not Pregnant)?
A: You may not be pregnant, or it may be too early to tell. If you do not get your period within a few days, you should retest with another FIRST RESPONSE™ EasyCup Pregnancy Test. It is possible that your expected period may be later than anticipated or your urine may not have had enough pregnancy hormone for the test to give a positive result. If upon retesting you get another negative result and your period has still not started, you should call your doctor.

Q: Do any drugs or medications affect the test?
A: Only medications containing the pregnancy hormone, human chorionic gonadotropin (hCG), can affect the result. The test should not be affected by hormone therapies containing Clomiphene citrate, alcohol, pain killers, antibiotics, or the contraceptive pill.

Q: Will the amount of liquid I drink affect the result?
A: Excessive fluid intake including water could dilute your urine, causing the amount of hCG present not to be detected properly. You should not drink too much liquid before testing.